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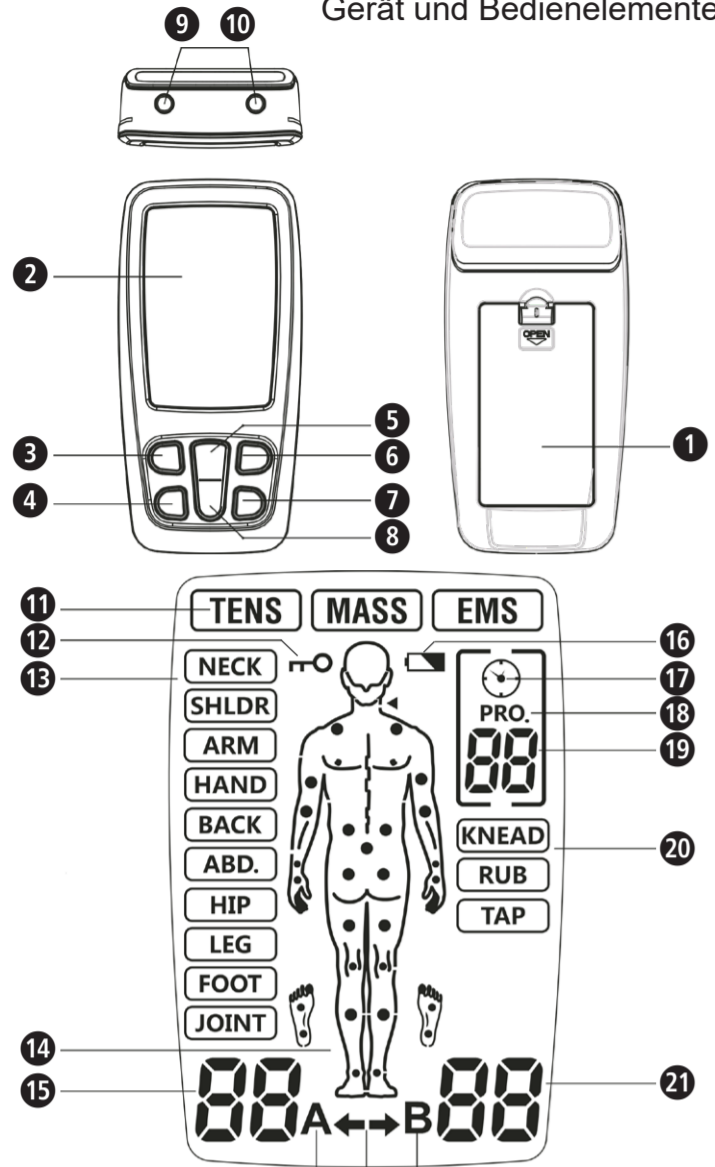
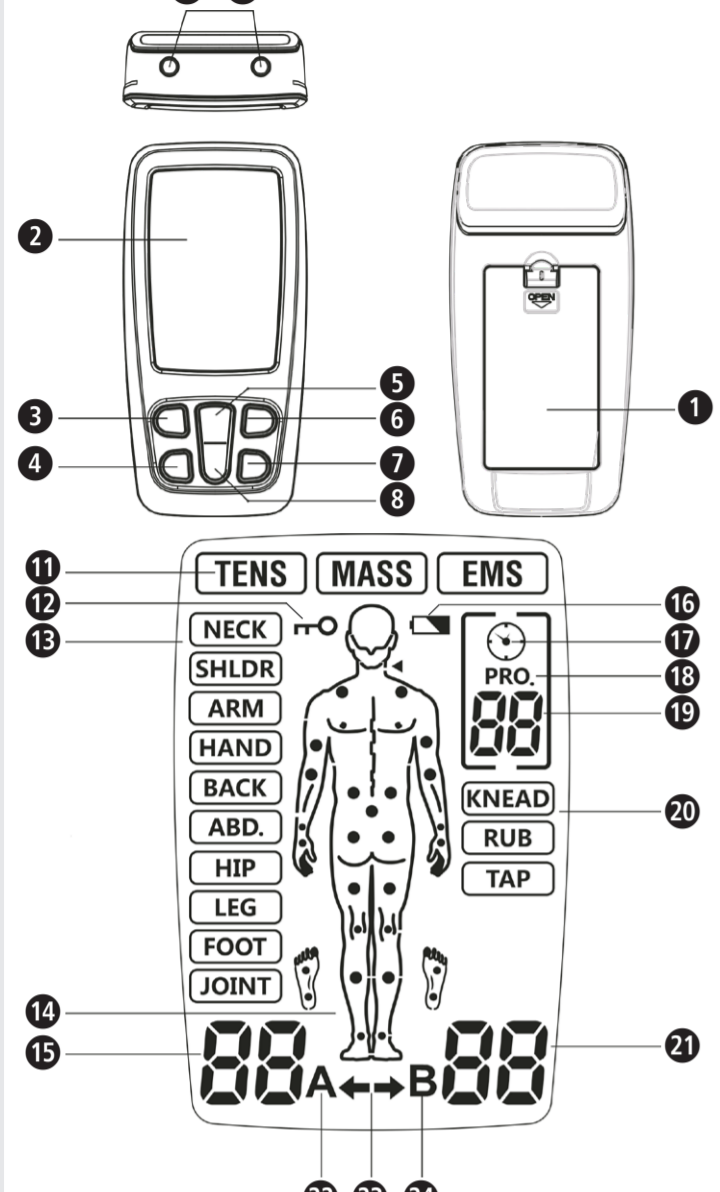
**Shenzhen Roundwhale Technology Co., Ltd.**  
202, 2/F, Building 27, Dafa Industrial Park, Longxi community,  
Longgang street, Longgang district, Shenzhen, China.  
Email: [info@roovjoy.com](mailto:info@roovjoy.com)  
Made in China

**Shanghai International Holding Corp. GmbH** (Europe)  
Eiffestraße 80, 20537 Hamburg, DEUTSCHLAND  
[shholding@hotmail.com](mailto:shholding@hotmail.com)

**GB Instructions for use**  
**3-in-1-Electrotherapy device TT 200**

Device and controls

**Explanation of symbols**

- Follow all instructions for use! This instruction manual belongs to this device. The instruction manual includes important information on the initial start-up and handling. Read this instruction manual completely. Failure to follow these instructions may result in serious injury or damage to the device.
- WARNING**  
These warnings must be followed to prevent possible injury to the user.
- CAUTION**  
These instructions must be followed to prevent possible injury to the device.
- NOTE**  
These instructions provide you with useful additional information regarding installation or operation.
- Users of an artificial pacemaker are prohibited from using the device!
- IP22**  
Information about protection type against foreign objects and against harmful effects due to the ingress of water.
- Disposal only according to the "WEEE" Waste Electrical and Electronic Equipment EC Directive.
- Recycling symbols / codes:  
These are used to provide information about the material and its proper use and recycling.
- Complies with the European Medical Device Regulation (EU) 2017/745.
- Catalogue Number
- Serial Number
- Medical Device
- Authorized representative in the European Community / European Union
- Unique Device Identifier
- Importer
- Manufacturer
- Fragile, handle with care
- Electric shock protection type: Class II equipment
- Batch code
- Device classification: type BF applied part
- Humidity range
- Ambient pressure limitation
- Temperature range
- Date of manufacture
- Keep dry

**GB IMPORTANT INFORMATION!**  
**KEEP IN A SAFE PLACE!**

**Please read this instruction manual carefully, in particular the safety instructions, before using the device. Keep the instruction manual in a safe place for later reference.**  
**If you pass the device on to a third party, this instruction manual must remain with the device.**

- Therapeutic sessions with the **medisana TT 200** do not replace medical diagnosis or treatment. Ask your doctor about each kind of pain or disease, before using the device.
- If your pain does not diminish, does not become significantly milder or last longer than 5 days, discontinue use and contact your doctor.
- Do not apply stimulation over your front and lateral neck because this could cause severe muscle spasms resulting in closure of your airway, difficulty in breathing or adverse effects on heart rhythm or blood pressure.
- Do not apply stimulation across your chest because the application of electrical current to the chest may cause rhythm disturbances to your heart, which could be lethal.
- Do not apply stimulation over - or in proximity to - cancerous lesions.
- Do not apply stimulation in the presence of electronic monitoring equipment (e.g., cardiac monitors, ECG alarms), which may not operate properly when electrical stimulation device is in use.
- Do not apply stimulation when in bath or shower.
- Do not apply stimulation while sleeping.
- Do not apply stimulation while driving, operating machinery or during any activity when electrical stimulation can put you at risk of injury.
- Apply stimulation only to normal, intact, clean, healthy skin.
- The long-term effects of electrical stimulation are unknown. Electrical stimulation device cannot replace drugs.
- Stimulation should not take place while the user is connected to high-frequency surgical equipment, which may cause burn injuries on the skin under the electrodes, as well as problems with the stimulator.
- Do not use the device in the vicinity of short or microwave therapeutic devices as they may affect the output power of the device.
- Never use it near the cardiac area. Stimulation electrodes should never be placed anywhere on the front of the thorax (marked by ribs and breastbone), but above all not on the two large pectoral muscles. There it can increase the risk of ventricular fibrillation and lead to cardiac arrest.
- Never use it on the eye, head and face area.
- Never use it near the genitals.
- Never use it on the areas of the skin which lack normal sensation.
- Keep electrodes separated during treatment. It could result in improper stimulation or skin burns if electrodes are in contact with each other.
- Keep the stimulator out of reach of children.
- The appliance is intended for use by adults only.
- The therapy should be pleasant. If you feel pain or feel the application is unpleasant, do not increase the intensity but stop the usage and consult your doctor.
- TENS is not effective for pain of central origin including headache. It is not a substitute for pain medications and other pain management therapies.
- TENS is a symptomatic treatment and, as such, suppresses the sensation of pain that would otherwise serve as a protective mechanism.
- Effectiveness is highly dependent upon the patient selection for a practitioner qualified in the management of pain patients.
- Since the effects of stimulation of the brain are unknown, stimulation should not be applied across your head, and electrodes should not be placed on opposite sides of your head.
- The safety of electrical stimulation during pregnancy has not been established.
- You may experience skin irritation or hypersensitivity due to the electrical stimulation or electrical conductive medium (silica gel).
- If you have suspected or diagnosed heart disease or epilepsy, you should follow precautions recommended by your physician.
- Use with caution if you have a tendency to bleed internally, e.g. following an injury of fracture or during your menstrual period.
- Consult with your physician prior to use the device after a recent surgical procedure, because stimulation may disrupt the healing process.
- Only one patient may use the device at a time.
- This stimulator should not be used by patients who is noncompliant and emotionally disturbed including dementia.
- Only use the device as described in this instruction manual. Any other use will be deemed as not in accordance with the instructions and may lead to damage to property or even injury to persons. **medisana** assumes no liability for damages incurred as the result of improper use. The device is designed for private use.
- Rare cases of skin irritation may occur at the site of the electrode placement following long-term application.
- Do not use this device in the presence of other equipment which sends electrical pulses to your body.
- Do not use sharp objects such as a pencil or ballpoint tip to operate the buttons on the control panel.
- Check the electrode connections before each use.
- Electrical stimulators should be used only with the electrodes recommended for use by the manufacturer.

**BATTERY SAFETY INFORMATION**

- Do not disassemble batteries!
- Clean the battery and device contacts if necessary before putting in the batteries!
- Remove discharged batteries from the device immediately!
- Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of this parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Keep the battery compartment well sealed!
- Remove the batteries from the device if it is not going to be used for an extended period!
- Keep batteries out of children's reach!
- Do not attempt to recharge these batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Keep unused batteries in their packaging away from metal objects in order to prevent short circuiting!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

**Intended use, Intended purpose**

The device is designed to be used for temporary relief of pain, including the acute and chronic pain relief. The device is designed to be used for stimulate muscles for prevention of muscular atrophy, muscle strengthening and improve, increasing local blood circulation and facilitate muscle performance.

**Target population**

The device using the object (patient) must be 18 years or older of adults.

**Intended user**

Medical staff or lay persons.

**Intended condition**

Intended for use in the home, hospital and health care facilities.

**Indications**

- It is used for temporary relief of pain associated with sore and aching muscles in the neck, shoulder, back, joint, hip, hand, abdomen, upper extremities (arm) and lower extremities (leg) due to strain from exercise or normal household work activities.
- Relaxation of muscle spasms
- Prevention or retardation of disuse atrophy
- Increasing local blood circulation
- Muscle re-education
- Immediate post-surgical stimulation of calf muscles to prevent venous thrombosis
- Maintaining or increasing range of motion

**Contraindications**

- Do not use this device if you have a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic device. Such use could cause electric shock, burns, electrical interference, or death.
- The device should not be used when cancerous lesions or other lesions are present in the treatment area.
- Stimulation should not be applied over open wounds or rashes, or over swollen, red, infected, or inflamed areas or skin eruptions (e.g. phlebitis, thrombophlebitis, varicose veins, arteriosclerosis obliterans etc.).
- Electrode placements must be avoided that apply current to the carotid sinus region (anterior neck) or transcerebrally (through the head).
- Apprehensive patients – Usage of electrical stimulation requires patient cooperation, hence the procedure shouldn't be attempted in patients with a communication handicap or a mental disability.
- Patients with cerebrovascular problems, patients with a history of aneurysm, stroke and transient ischaemia shouldn't be treated using electrical stimulation, as it stimulates peripheral blood flow which can be fatal in such cases.
- Epileptic patients – Electrical stimulation "pulses" have the potential to trigger a seizure.
- Acute pain cases/pain of unknown etiology – Usage of TENS in undiagnosed cases may hinder in the diagnosis.
- Do not use in pregnancy, especially in the first trimester.

**Side effects**

- Skin irritation or burns can occur in the area of the electrode application.
- In very rare cases, first-time users of an EMS device have reported drowsiness or fainting. We recommend that you use the device while sitting until you get used to its use.
- If you feel uncomfortable, reduce the stimulation intensity to a comfortable level. Contact your doctor if the problems persist.
- Possible allergic skin reaction to tape or gel may occur.
- If symptoms of tachycardia and extrasystolia (rapid heartbeat or extra stimulation) appear during treatment, stop the treatment and seek medical attention immediately.

**Scope of delivery**

First, check whether the device is complete and is not damaged. If in doubt, do not operate the device and contact your supplier or service centre.

- The contents include:
- 1 **medisana 3-in-1-Electrotherapy device TT 200**
  - 4 Elektrodes 50 x 50 mm
  - 2 Connection cables
  - 3 Batteries (Type AAA, LR03, 1.5 V)
  - 1 Instructions for use

If you notice any transport damage while unpacking, please contact your supplier immediately.

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

**WARNING**  
Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

**Device and controls**

- 1 Battery compartment
- 2 Display field
- 3 On/Off/M button
- 4 T button
- 5 "+" key
- 6 B button
- 7 CH button
- 8 "-" button
- 9 Output port
- 10 Therapy mode
- 11 Device locked
- 12 Body region
- 13 Recommended treatment points
- 14 Intensity Channel A
- 15 Battery low
- 16 Timer symbol
- 17 Program symbol
- 18 Program no. / Therapy time
- 19 Massage mode
- 20 Intensity Channel B
- 21 Symbol for Channel A
- 22 Channel indicator
- 23 Symbol for Channel B

**Insert / replace batteries**

Open the cover of the battery compartment on the back of the device and insert the enclosed 3 batteries AAA 1.5V according to the symbols (+/-). Close the battery compartment again. When changing the batteries later on, only use 1.5V batteries of type AAA (Micro/LR6).

**Cable connection**

Make sure that the device is switched off before you connect the cable. First connect the electrodes with the two connection cables and then the cables with the device. On the upper side of the device there are two output connections 9 for connecting the electrode lines. The device has two connections for two channels (A and B). Depending on your needs, you can decide whether you want to use only one channel with one pair of electrodes or connect both channels with two pairs of electrodes. With two pairs of electrodes you can stimulate two different body regions at the same time.

**WARNING**  
**NEVER insert the electrode cables into an electrical outlet or any other power source!**

**Applying the electrodes**

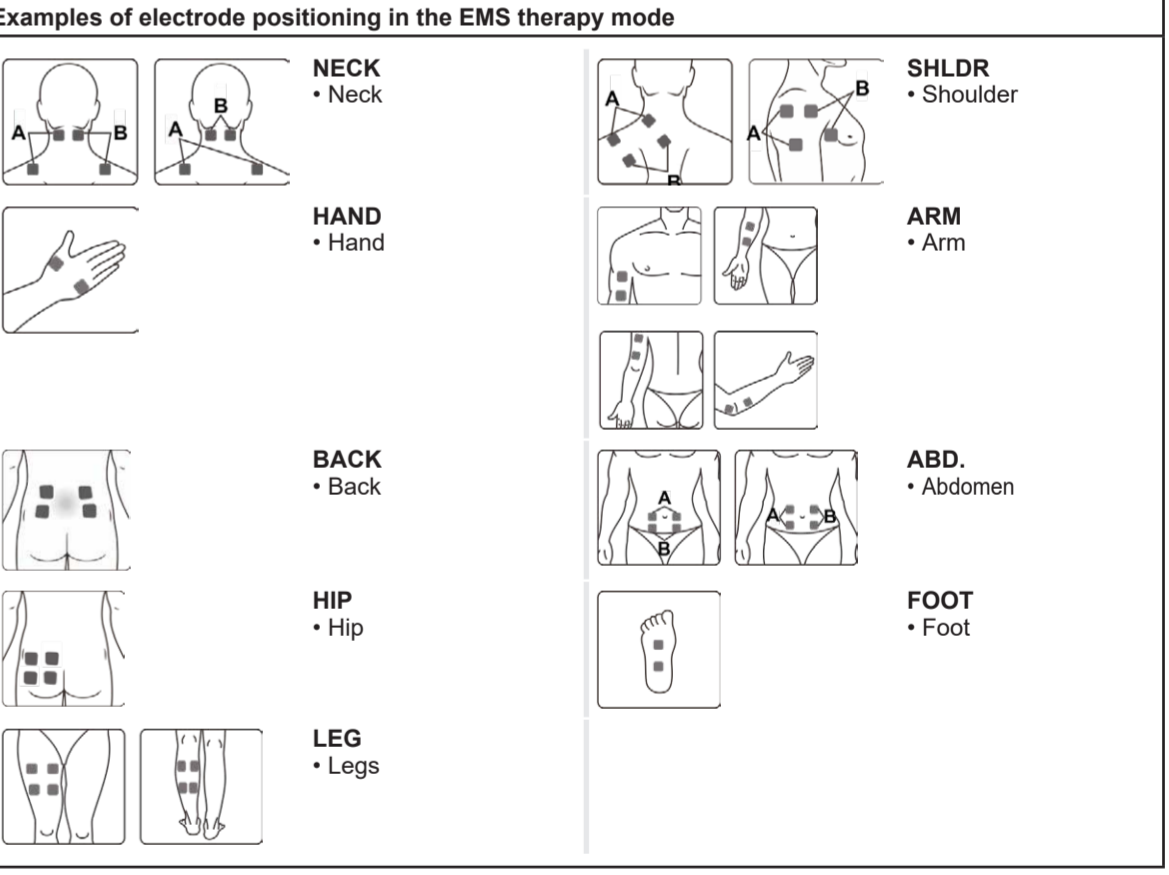
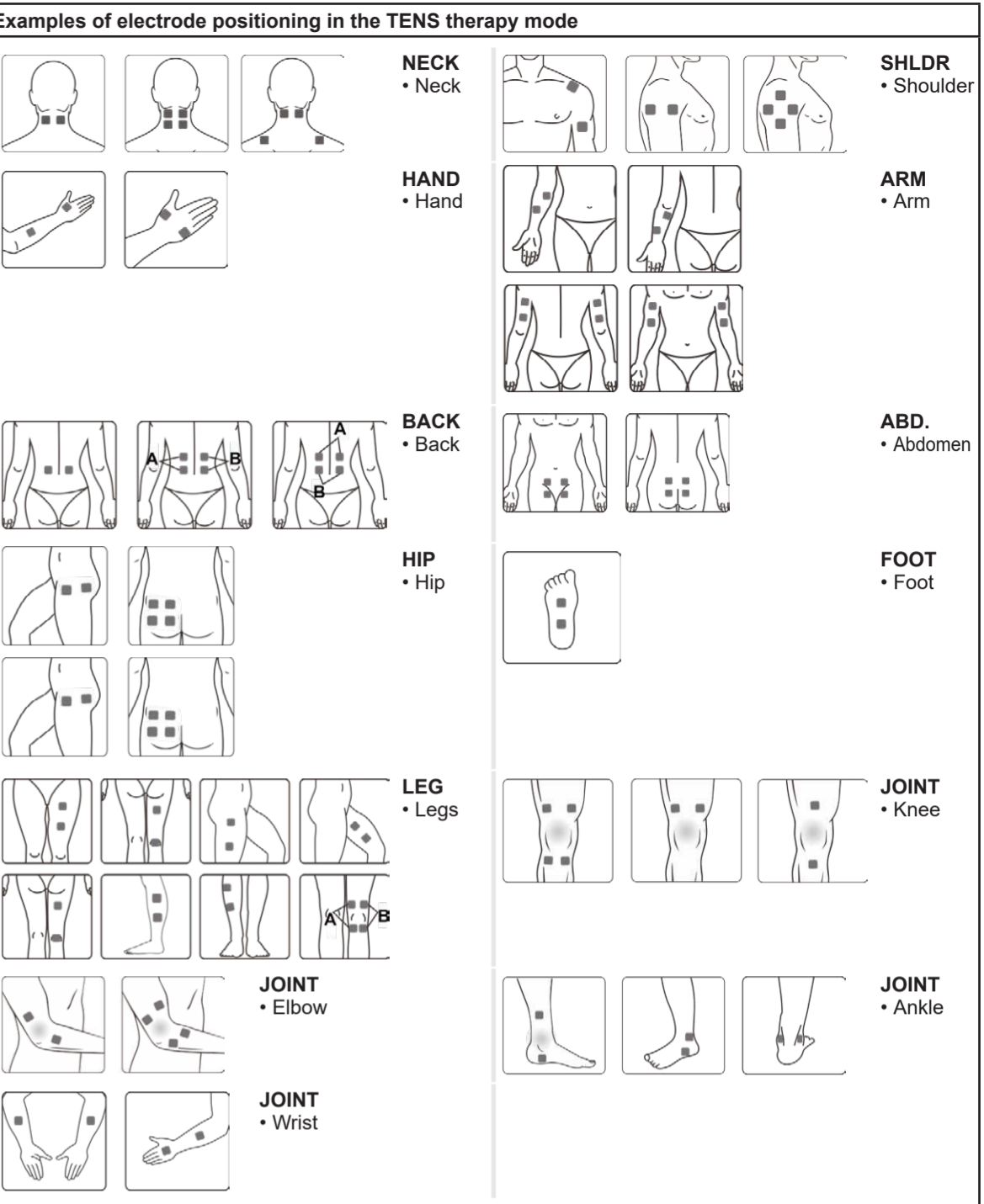
The reusable carrier film can be removed immediately before use - keep it for later use. Then place the electrodes according to your pain area (follow the instructions of your doctor).

**WARNING**

- **The device must be switched off when applying the electrodes!**
- **Remove any skin cream or ointment residue before applying the electrodes. The skin must be fat-free and dry.**
- **Do not stick the electrodes on injured or inflamed skin areas (wounds, pimples, rashes, redness, etc.).**
- **Always apply the electrodes according to your doctor's instructions and only then switch on the device.**
- **Make sure that the electrodes cannot fall off during the therapy. This can lead to an unintended electric shock!**
- **Switch off the device immediately if an electrode detaches from the skin during treatment.**

In the following you will find placement suggestions for the electrodes depending on the therapy mode. However, please note that each patient reacts differently to stimulation current therapies - therefore the positioning of the electrodes and the program selection for your requirements may differ from the therapies presented here. You should make a note of the optimum parameters for you (electrode position, program and current) for subsequent treatments.


The recommended treatment points 13 for each program are shown in the illustration. The following illustrations show an example of how to attach the electrodes to these treatment points.

## Switching on


To switch on, briefly press the On/Off/M button **1**. The indicator lights up and the device is in standby mode. Now select the therapy mode using the On/Off/ M button **2**:

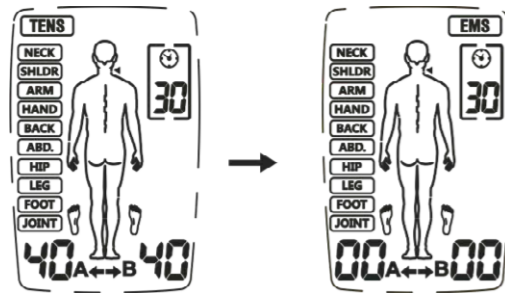TENS : for pain relief  
(TENS = Transcutaneous Electrical Nerve Stimulation)


MASS : Massage to relax tense muscles

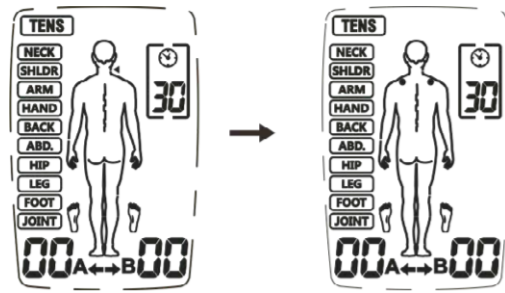EMS : For muscle building  
(EMS = Electrical Muscle Stimulation)

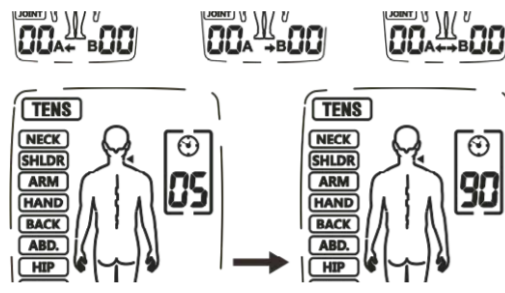
Then use the B button **3** to select the area of the body to be stimulated:

Now use the CH button **4** to select the channel (Channel A, Channel B or both channels):


Adjust the treatment time with the T-button **5**:

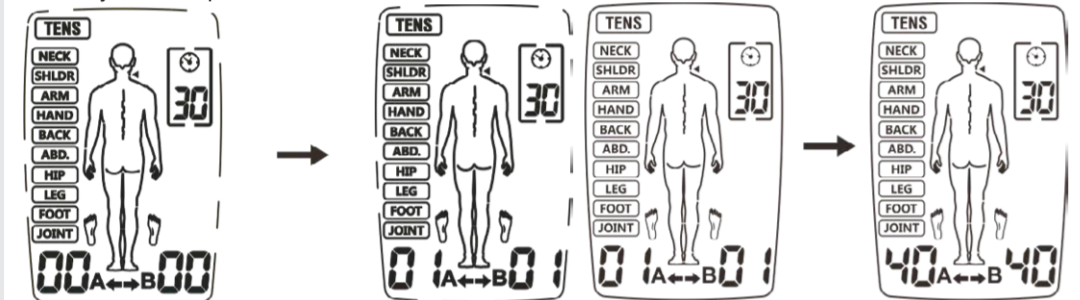Now press the + button **6** to increase the intensity of the selected channel. The corresponding intensity level is shown on the display. Each time you press the + button **6**, you can increase the intensity further up to the maximum level 40:


With the - button **7** the intensity is reduced accordingly. If you reduce the intensity to 0, the device switches to standby mode.

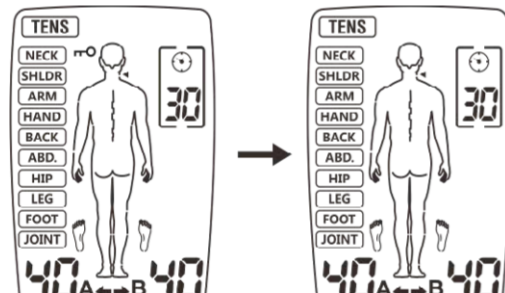During use, you can lock the device by pressing and holding the B button **8**. The symbol **9** then appears on the display. You cannot accidentally adjust the intensity while locking. To release the lock, press and hold the B button **8** until the symbol **9** disappears.

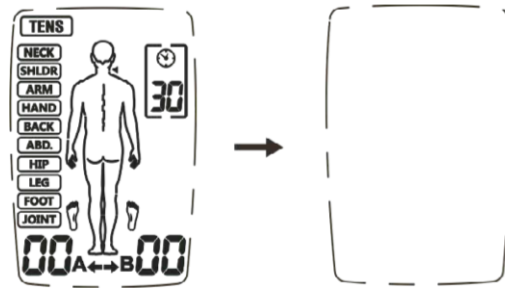Press the On/Off/M button **1** to exit the application. Pressing the On/Off/M button **1** again for a long time switches the device off.

The device has an automatic load detection at all set intensities above level 5. If the electrodes do not stick to the skin properly or there is another connection problem, the device automatically switches to level 0 and the "A" or "B" symbol flashes. The device then switches back to standby mode.

If the battery voltage is too low, the "Battery low" symbol **10** flashes. Stop the application and replace the batteries.

## Use of the electrodes

- The electrodes supplied may only be used with the medisana 3-in-1 Electrotherapy device TT 200. Make sure that the device is switched off before applying and removing the electrodes.
- If you want to reposition the electrodes during use, switch off the device first.
- Use of the electrodes can cause skin irritation. If you should notice this (e.g. red skin, blistering or itching) stop the use. Do not use the electrodes regularly on the same skin areas.
- Never use the electrodes together with other people.
- The electrodes must be in full contact with the skin to avoid possible "hot spots", which can cause burns.
- Do not use the same electrodes more than 10 times because the connection to the skin surface deteriorates with each use.
- The adhesive force of the electrodes depends on the skin properties, the storage condition and the number of applications. If your electrode pads no longer stick completely to the surface of the skin, replace them with new ones. After use, stick the electrode pads back onto the protective film and keep the electrodes in the storage bag so that they do not dry out. As a result, the adhesive force is retained over a longer period of time.
- Clean and dry the skin areas where the electrodes are to be applied.
- Never remove the electrodes from the skin while the device is still in operation.
- Only use original electrodes that are recommended by the manufacturer. Other electrodes could cause injuries.
- Each person reacts differently to electrical nerve stimulation. There may be individual deviations in the correct placement of the electrodes. Contact your doctor to find out which placement is best for you.
- Do not use electrodes smaller than the ones provided. Such use could make the current density too high and cause injury.
- Do not change the electrode pads supplied, e.g. by cutting.
- Make sure that the area from which the pain originates is completely covered with the electrode pads. In the case of painful muscle groups, all affected muscles must be reached by the electrodes.
- Do not use detergent to clean the electrode pads before and after use to avoid damaging the adhesion of the electrode pads.
- The electrode pads must always be touched with clean hands, it is recommended to replace the electrode pads if they become dirty.

## NOTES on the TENS application

- If you find the output intensity too strong, you can reduce it with the „-“ button **7**;
- As long as you feel comfortable during the application, continue it to the end. Pain improvement usually occurs after about 5 to 10 minutes;
- We recommend 1 to 2 applications per day for about 1 week;
- If there is no significant improvement in pain after this time, contact your doctor.

## NOTES on the EMS application

- Pay attention to the correct placement of the electrodes as described in these instructions;
- We recommend 1 to 2 applications per day for about 1 week;
- If you feel uncomfortable during the application, pause the application and/or reduce the intensity.

## Maintenance and cleaning

- In order to ensure trouble-free and long-term functional reliability of the device, the following points should be observed:
- Remove the electrodes from the device and clean the device with a soft, slightly damp cloth. In the case of heavy soiling, you can also use a mild detergent.
- Protect the device from moisture. Do not hold the device under running water and do not immerse it in water or other liquids.
- Do not place the device on hot surfaces and do not expose it to direct sunlight.
- Clean the surface of the electrode pads with a slightly damp cloth. Make sure that the device has been switched off beforehand!
- For hygienic reasons, each user should use their own electrode pads.
- Do not use aggressive substances or chemical cleaners to clean the device.
- Make sure that no liquid gets into the device. Do not use the device again until it is completely dry.
- Never clean the device during operation. Before cleaning, make sure that the device is switched off and the batteries have been removed!

## Maintenance

- The manufacturer cannot be held responsible for damage caused by unauthorised repairs. If your device has a technical problem, contact customer service.
- Do not try to carry out repairs yourself.
- Opening the device will render the warranty void.
- Your device was tested before being sold and does not require calibration or regular maintenance. If you have any questions, please contact customer service.

## Storage

Switch off the device and remove the cables. Place the electrodes on the carrier film and pack them together with the device and the cables in the original packaging. Observe the storage conditions in the technical data! If you do not use the device for a long time, remove the batteries from the battery compartment.

## Troubleshooting

| Problem                                 | Possible causes   | Countermeasures  |
|---|---|--|
| No display after changing the battery   | <ul style="list-style-type: none"> <li>Batteries not inserted correctly or foreign objects in the battery compartment.</li> <li>Battery type may be wrong.</li> </ul>                                     | <ul style="list-style-type: none"> <li>Check and clean the battery compartment.</li> <li>Insert new batteries of the correct type. Pay attention to the correct polarity!</li> </ul>   |
| No or too weak stimulation              | <ul style="list-style-type: none"> <li>Electrodes not placed correctly on the skin</li> <li>Connection to the device is not ideal</li> <li>Used batteries</li> <li>Too dry skin</li> </ul>                | <ul style="list-style-type: none"> <li>Check that the electrodes are correctly attached to the skin and that the cable connection to the device is optimum.</li> <li>Replace the battery.</li> <li>Wipe your skin and the electrodes with a damp cotton cloth.</li> </ul>  |
| The skin tickles or burns during use    | <ul style="list-style-type: none"> <li>The application is too long.</li> <li>Electrodes are not placed correctly on the skin or the contact is not optimum</li> <li>The skin is hypersensitive</li> </ul> | <ul style="list-style-type: none"> <li>Use the device once a day and shorten the time of use.</li> <li>Check that the electrodes are correctly attached to the skin.</li> <li>Wipe your skin and the electrodes with a damp cotton cloth.</li> <li>Consult your doctor if you have hypersensitive skin.</li> </ul> |
| The device stops during the application | <ul style="list-style-type: none"> <li>Electrodes are not placed correctly on the skin or the contact is not optimum</li> <li>Used batteries</li> </ul>   | <ul style="list-style-type: none"> <li>Check that the electrodes are correctly attached to the skin.</li> <li>Replace the battery.</li> </ul>  |

## Program tables

| Mode    | Body part | Frequency Hz      | Impulse width µs | Duration min.                    | Waveform           |
|---------|-----------|-------------------|------------------|----------------------------------|--------------------|
| TENS    | Neck      | 80-120            | 120-100          | Standard: 30<br>Adjustable: 5-90 | Modulation         |
|         | Shoulder  | 80-100            | 100              |                                  | Modulation         |
|         | Arm       | 2-60              | 260-160          |                                  | Modulation         |
|         | Hand      | 100               | 100              |                                  | Consistent         |
|         | Back      | 60/50/45/10/50/35 | 200              |                                  | Modulation         |
|         | Abdomen   | 120               | 55               |                                  | Consistent         |
|         | Hips      | 100               | 150              |                                  | Short Pulse groups |
|         | Leg       | 40/6/50           | 250              |                                  | Modulation         |
|         | Foot      | 80-120            | 100-120          |                                  | Modulation         |
|         | Joints    | 120               | 100-120          |                                  | Modulation         |
| EMS     | Neck      | 30                | 200              |                                  | Synchronous        |
|         | Shoulder  | 45                | 200              |                                  | Synchronous        |
|         | Arm       | 50                | 150              |                                  | Synchronous        |
|         | Hand      | 4                 | 200              |                                  | Synchronous        |
|         | Back      | 60                | 200              |                                  | Synchronous        |
|         | Abdomen   | 20                | 200              |                                  | Synchronous        |
|         | Hips      | 30                | 150              |                                  | Synchronous        |
|         | Leg       | 80                | 200              |                                  | Synchronous        |
| Massage | Knead     | 28-44             | 120-250          |                                  | Modulation         |
|         | Rub       | 25-79             | 120-250          |                                  | Modulation         |
|         | Tap       | 49-97             | 100-240          |                                  | Modulation         |

## Technical information

|   |  |
|---|--|
| Name and model                          | 3 × 1.5V=, Micro batteries (AAA, LR03) 2   |
| Power supply                            | Biphasic rectangular pulse   |
| Channels                                | Max. 120mA (at 500 Ohm load)   |
| Waveform                                | adjustable in 40 steps   |
| Output current                          | ±20% for all parameters  |
| Intensity                               | TENS, EMS and MASSAGE  |
| Output precision                        | TENS: 10; EMS: 9 MASSAGE: 3  |
| Application modes                       | 55 – 260 µs, depending on the program  |
| Number of programs                      | 2 – 120 Hz, depending on the program   |
| Impulse width                           | 5 – 90 minutes, adjustable depending on the program after 1 minute                   |
| Frequency                               | +5°C to +40°C at 15% to 93% rel. humidity: atmospheric pressure 700 hPa to 1060 hPa  |
| Application time                        | -10°C to +55°C at 10% to 95% rel. humidity: atmospheric pressure 700 hPa to 1060 hPa |
| Automatic switch-off                    | approx. 109 × 54.5 × 23 mm   |
| Operating conditions                    | approx. 50 × 50 mm   |
| Storage conditions                      | approx. 76 g   |
| Dimensions                              | 5 years  |
| Size of electrode pads                  | Self-adhesive Electrode pads shelf life  |
| Weight                                  | 3 years  |
| Device service life                     | 88347  |
| Self-adhesive Electrode pads shelf life | 4015588 88347 7  |
| Item number                             |  |
| EAN number                              |  |

## Replacement parts:

- Cable set - Item number 88359 / EAN 4015588 88359 0
- 8 self-adhesive electrodes - Item number 88349 / EAN 4015588 88349 1

## Disposal — Waste Electrical and Electronic Equipment (WEEE)

This product is subject to European Directive 2012/19/EU on waste electrical and electronic equipment and is marked accordingly. Never dispose of electronic devices with household waste. Please seek out information about the local regulations with regard to the correct disposal of electrical and electronic products. Correct disposal helps to protect the environment and human health. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your local authority or your supplier for information about disposal.

## Warranty and repair terms

Your statutory warranty rights are not restricted by our guarantee below. Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

- The warranty period for **medisana** products is 3 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
  - All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - Applied parts which are subject to normal wear and tear.
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

**In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.**

The current version of this instruction manual can be found under <https://docs.medisana.com/88347>.

Information about service can be found here: <https://www.medisana.com/servicepartners>

**medisana GmbH**,  
Carl-Schurz-Str. 2, 41460 NEUSS, GERMANY

**Shenzhen Roundwhale Technology Co., Ltd.**  
202, 2/F, Building 27, Dafa Industrial Park, Longxi community,  
Longgang street, Longgang district, Shenzhen, China.  
Email: [info@roovjoy.com](mailto:info@roovjoy.com)  
Made in China

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**Shanghai International Holding Corp. GmbH (Europe)**